


CLEANING COMMOMODES

A COMPLETE STEP-BY-STEP APPROACH



Put on required PPE for cleaning & disinfection according to hospital policy.

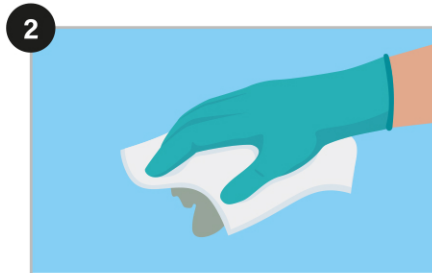


Before wiping down, remove any gross contamination with Incidin OxyWipe S.

USE INCIDIN OXYWIPE S



As many as required.

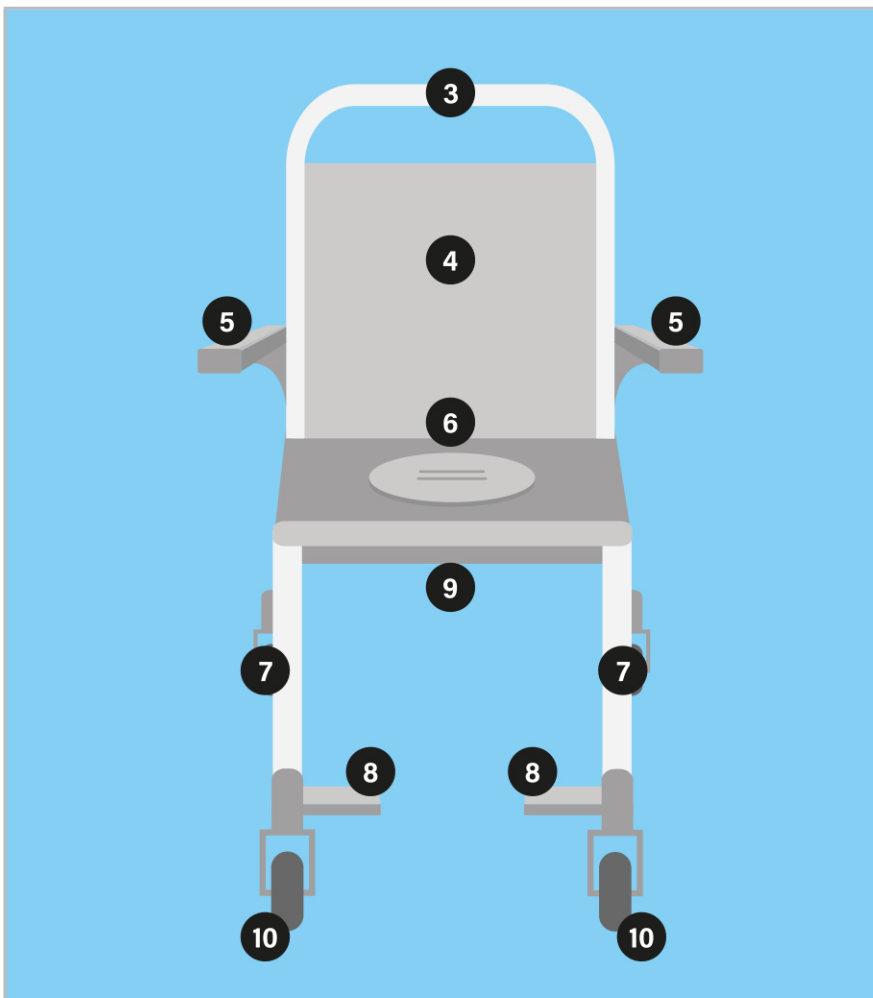

CLEANING COMMOMODE

3. Wipe handle.
4. Wipe back plate.
5. Wipe arms, including underside.
6. Detach seat or raise and clean all parts. Reassemble. Dispose of wipe and select a new wipe.
7. Wipe legs, tip up commode.
8. Wipe frame.
9. Wipe underneath the seat;
10. Wipe wheels;
11. Dry commode;



Remove PPE and wash your hands